

Battery Pack

SB-752P

General

The SB-752P battery pack is a rechargeable battery for SVM-7500 series bedside monitor.

Thoroughly read this manual and the operator's manual of the connected monitor before use.

Safety Information

- ⚠ DANGER:** A danger alerts the user to a hazardous situation which causes death or serious injury.
- ⚠ WARNING:** A warning alerts the user to possible injury or death associated with the use or misuse of the instrument.
- ⚠ CAUTION:** A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ WARNING

To avoid leakage, overheating, explosion, fire or skin burn:

- Do not give strong impact to or deform the battery.
- Do not disassemble or modify the battery.
- Do not put the battery pack into fire or heat it.
- Do not use the battery in unspecified instruments.
- Do not use the battery pack at temperatures over 40 °C (104 °F).

⚠ WARNING

Failure to observe any of the following may cause battery malfunction, overheating, explosion, fire and skin burn.

- Do not immerse the battery pack in liquid or get it wet.
- Do not leave the battery near a heat source such as a stove.
- Do not short-circuit the + and – terminals on the battery.
- Do not charge the battery in a high temperature place such as near a stove or in sun-heated cars.

⚠ WARNING

If the battery pack is damaged and the substance inside the battery pack contacts the eyes, wash immediately and thoroughly with water and see a physician. Never rub your eyes, because you may lose your eyesight.

⚠ CAUTION

Do not leave the battery pack in reach of patients.

⚠ CAUTION

If the battery pack is damaged and the substance inside the battery pack contacts the skin or cloth, wash immediately with clear water. The skin may get irritated.

⚠ CAUTION

Do not pull out the battery connector forcibly. Otherwise, the battery connector may break and the battery pack may become unusable.

⚠ CAUTION

The battery connection cable must be on the right side. Do not insert the battery pack in the wrong direction.

NOTE: Read the SVM-7500 series bedside monitor operator's manual for handling and safety of the battery pack.

- NOTE**
- When using a new battery pack, fully charge it before use.
 - Replace the battery pack with a new one every year. The battery pack lifetime may become shorter depending on the operating environment.
 - When bringing the battery pack on an airplane, consult the airline beforehand.
 - When transporting the battery pack on an airplane, consult the shipping company beforehand.

- NOTE**
- New battery packs are not charged at the factory. Charge the battery pack before use.
 - The battery pack lifetime may become shorter depending on the operating environment.
 - Charge the battery pack every three months and whenever indicated by the remaining battery indication and messages.

Disposal

⚠ CAUTION

Dispose of Shanghai Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.



Manufacturer

SHANGHAI KOHDEN MEDICAL
ELECTRONIC INSTRUMENT CORP.
No.567 Huancheng Bei Road, Shanghai Comprehensive
Industrial Development Zone, Fengxian District,
Shanghai 201401, China
Phone +86 21-5743-6998 Fax +86 21-5743-6939
<https://www.nihonkohden.com/>



European Representative

NIHON KOHDEN EUROPE GmbH
Raiffeisenstrasse 10, D-61191 Rosbach, Germany
Phone +49 6003-827-0 Fax +49 6003-827-599