

Headband for EEG

BE-090AS, BE-090AM

General

The BE-090AS and BE-090AM headbands are for attaching electrodes for EEG monitoring. The headband is used mainly in an ER or ICU.

Safety Information

⚠ CAUTION A caution alerts the user to possible injury or problems with the instrument associated with its use or misuse such as instrument malfunction, instrument failure, damage to the instrument, or damage to other property.

Pay attention to all safety information in the Operator's Manual or Installation Guide.

⚠ CAUTION

Do not attach the headband on a wound or inflamed skin. It may cause irritation or redness on the skin.

⚠ CAUTION

If the skin gets irritated or redness appears on the skin from the headband, stop using the headband. Take extreme care for patients with delicate skin.

⚠ CAUTION

Check the patient's scalp every hour. Irritation or redness may occur.

⚠ CAUTION

Use the headband with the appropriate size for the patient's head. If the headband does not fit the head properly, measurement may be incorrect.

⚠ CAUTION

Do not alter the shape of the headband. EEG might not be recorded.

⚠ CAUTION

Do not spill any liquids on the headband during measurement. Measurement may be incorrect.

⚠ CAUTION

Do not use a deteriorated headband. If the headband is deteriorated, replace it with a new one. If a deteriorated headband is used, measurement may be incorrect.

⚠ CAUTION

Do not attach the headband to the patient in the lateral recumbent position because it puts too much pressure on the head.

⚠ CAUTION

Dispose of Nihon Kohden products according to your local laws and your facility's guidelines for waste disposal. Otherwise, it may affect the environment. If there is a possibility that the product may have been contaminated with infection, dispose of it as medical waste according to your local laws and your facility's guidelines for medical waste. Otherwise, it may cause infection.

⚠ CAUTION

- Wash the headband if it is dirty. Thoroughly rinse the headband after washing it with neutral detergent.
- If an infectious substance such as blood is on the headband, dispose of the headband.

⚠ CAUTION

Do not sterilize this headband.

This Safety and Performance Information is an extract from the general and safety information sections of the most recent edition of Operator's Manual or Installation Guide. Therefore, the contents of your Operator's Manual or Installation Guide may differ from those of this Safety and Performance Information. For detailed operating procedures, follow the instructions of your Operator's Manual or Installation Guide.

 Manufacturer
NIHON KOHDEN CORPORATION
 1-31-4 Nishiochiai, Shinjuku-ku,
 Tokyo 161-8560, Japan
 Phone +81 3-5996-8041
<https://www.nihonkohden.com/>

EC REP European Representative
NIHON KOHDEN EUROPE GmbH
 Raiffeisenstrasse 10, 61191 Rosbach, Germany
 Phone +49 6003-827-0 Fax +49 6003-827-599

1st Edition: 24 Apr 2019
 2nd Edition: 20 Sep 2023

